


INSTRUCTIONS FOR USE

ZURNAL™ (zur nye) (nalmeffene injection)

The Instructions for Use contains information on how to give ZURNAL in response to a known or suspected opioid overdose in adults and children 12 years of age and older. You and your family members or caregivers should read the Instructions for Use that comes with ZURNAL before using it. Talk to your healthcare provider if you and your family members or caregivers have any questions about the use of ZURNAL.

ZURNAL auto-injector parts:

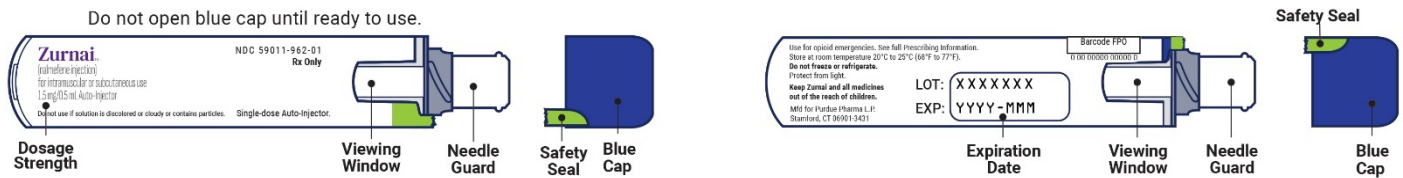

Important information you need to know before injecting ZURNAL

- The ZURNAL auto-injector is ready to use. **Do not** test or prime ZURNAL before use.
- Each ZURNAL auto-injector contains 1 dose of medicine and cannot be reused.
- Inject ZURNAL into the muscle (intramuscular) or underneath the skin (subcutaneous) of the outer thigh. **Do not** inject into any other area of the body.


Preparing to inject ZURNAL

- **Check person for signs of a suspected overdose:**
 - will not wake to voice or touch
 - very sleepy
 - not breathing well
- **Check the auto-injector label** to make sure you are injecting ZURNAL.
- **Check for any visible damage** to the auto-injector before using.
- **Check the auto-injector** to make sure the safety seal is not broken and the blue cap is secure and not missing.
- **Check the liquid through the viewing window** before using. It should be clear, colorless to light-yellow and free of particles. You may see air bubbles. This is normal.

Injecting ZURNAL

1. Remove the blue cap

- Twist the blue cap counterclockwise to break the safety seal and pull straight off to remove the cap.
- After the blue cap is removed, a few drops of liquid may appear on the needle end. This is normal.
- ZURNAL must be used right away after the safety seal is broken and blue cap is removed. **Do not** replace the blue cap after it has been removed. If you do not use ZURNAL right away, see Step 5 for instructions on how to throw away ZURNAL.
- **Do not** touch the needle end of the auto-injector after the blue cap is removed to prevent injury or accidental activation.

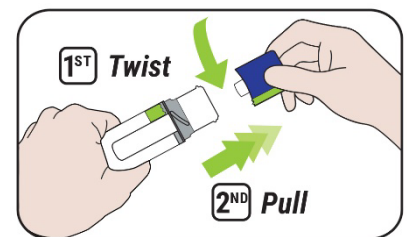


2. Position and inject

- Place the **needle guard** of the auto-injector against the **outer thigh**. The needle will inject through clothing.
- Push the auto-injector firmly down against the outer thigh until you hear a **“click”** (the click indicates the start of the injection).
- **Hold** in place for **3 seconds**.




3. Inspect the viewing window

- After holding in place for 3 seconds, the viewing window should turn completely orange. A fully blocked (completely orange) window confirms the injection is complete.
- Remove the auto-injector from the outer thigh.
- **Do not** replace the blue cap after the injection.
- If the viewing window is not fully blocked after injecting, repeat **Steps 1 to 3** using a new ZURNAL auto-injector.

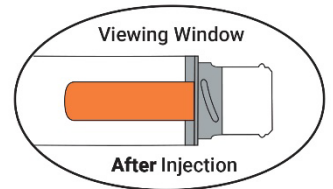

4. Call 911 and watch the person

- **Call 911** or get emergency medical help right away after the injection.
- Move the person on their side (recovery position).
 - Hand is supporting their head.
 - Bend their knee.
 - Turn their face to the side.
- **Watch** the person for response. **Wait for 2 to 5 minutes** to see if the person wakes up.
- **If no response**, repeat **Steps 1 to 3** using a new ZURNAL auto-injector to give another dose.
- If more ZURNAL auto-injectors are available, Steps 1 to 3 may be repeated every 2 to 5 minutes until the person responds or emergency medical help arrives.
- **Stay** with the person until medical help arrives, even if the person wakes up.

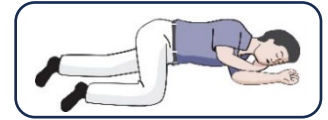


Wait 2 to 5 minutes



5. Throw away (dispose of) the used ZURNAL auto-injector

Disposing of (throwing away) ZURNAL

- Put your used ZURNAL auto-injector in a FDA-cleared sharps disposal container right away after use. Do not throw away (dispose of) ZURNAL in your household trash.
- If you do not have an FDA-cleared sharps disposal container, you may use a household container that is:
 - made of a heavy-duty plastic,
 - can be closed with a tight-fitting, puncture-resistant lid, without sharps being able to come out,
 - upright and stable during use,
 - leak resistant, and
 - properly labeled to warn of hazardous waste inside the container.
- When your sharps disposal container is almost full, you will need to follow your community guidelines for the right way to dispose of your sharps disposal container. There may be state or local laws about how you should throw away used needles and syringes. For more information about safe sharps disposal, and for specific information about sharps disposal in the state that you live in, go to the FDA's website at: <http://www.fda.gov/safesharpsdisposal>.
- Do not dispose of your used sharps disposal container in your household trash unless your community guidelines permit this. Do not recycle your used sharps disposal container.

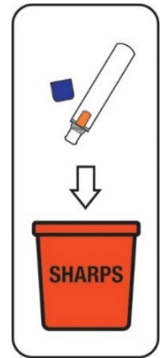

Important: Always keep the sharps disposal container out of the reach of children.

Storing ZURNAL

- Store ZURNAL at room temperature between 68°F to 77°F (20°C to 25°C).
- Do not freeze or refrigerate.
- Store ZURNAL in a clean dry place. Protect from light.
- Keep ZURNAL in the carton until use.
- During storage, check ZURNAL through the viewing window of the auto-injector every 30 days. The liquid should be clear, colorless to light yellow. If the ZURNAL liquid is discolored, cloudy, or contains solid particles, replace it with a new ZURNAL.
- Check the expiration date. Replace ZURNAL before the expiration date.

Keep ZURNAL and all medicines out of the reach of children.

Manufactured for Purdue Pharma L.P., Stamford CT, 06901-3431.

©2024, Purdue Pharma L.P., Stamford CT 06901-3431

For more information go to www.ZURNAL.com or call 1-888-726-7535.

This Instructions for Use has been approved by the U.S. Food and Drug Administration.

Issued: 08/2024